



Scott Airway Management is an **Australian** company that identified a problem. That **problem** is twofold –

1. People with beards are difficult to bag mask ventilate
2. Our healthcare professionals need to be protected from patients' aerosolised secretions during bag mask ventilation.

The **solution** is the **SAM SAFETY SEAL** for Guedel.



SAFETY SEAL
To Suit SAM GUEDEL AIRWAY

SINGLE
USE ONLY



SAFETY SEAL
To Suit SAM GUEDEL AIRWAY

SINGLE
USE ONLY

Patients must meet the following criteria –


- Suit size 9(yellow) or 10 (red) Guedel Airway
- Anaesthetised & Paralysed
- Have a beard

1



The SAM Safety Seal is provided with either a size 9cm or 10cm SAM Guedel airway. Ensure that the guedel is inserted into the SAM Safety Seal before use.

2

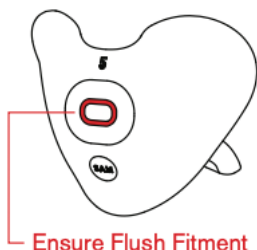

The guedel mouthpiece should fit flush into the SAM Safety Seal opening.

3



The SAM Safety Seal is now ready to use. Invert the shield to place the guedel into the patients' airway. Please complete an evaluation form after use.

